

Instructions for use

Product description:

Debrisoft® consists of polyester fibres, the reverse side is coated with polyacrylate. The product is individually packed and sterile (heat treated). Debrisoft® is for single use only and must not be re-sterilized. The product is sterile as long as the packaging remains unopened and undamaged.

Indications:

In wound bed preparation, Debrisoft® is intended as a rapid, highly effective and safe debridement method in the treatment of superficial wounds and the surrounding skin, for example in cases of

- diabetic ulcers
- arterial and venous ulcers
- pressure ulcers
- postoperative wounds healing by second intention

Debrisoft® can also be used for absorbing exudate, cellular debris and keratosis during debridement. Debrisoft® is gentle on intact tissue.

Please note:

There are also very good results with Debrisoft® in the treatment of wounds with major keratotic and necrotic coatings, provided these wounds have previously been treated with autolytic debridement.

Adverse reactions:

None known to date.

Application:

1. Remove the existing wound dressing.
2. Debridement procedure:
 - a) Soak Debrisoft® with e.g. water or saline, according to local guidelines.
 - b) Wipe the wound surface gently without pressure using the soft fibre side of the thoroughly moistened Debrisoft® (recommended quantity to moisten: approx. 20-40 ml).
 - c) If required, cleanse the peri wound skin with another thoroughly moistened Debrisoft®.
3. Following use, standard treatment for the condition should be continued.

Precautions:

- Debrisoft® must not be left on the wound.
- The product must not be used in case of known intolerance or an allergy against one or more of its constituents.
- Debrisoft® must not be cut to size